


RapidVet®-H

Canine Blood Typing

Canine Interpretation Guide

Positive for Agglutination

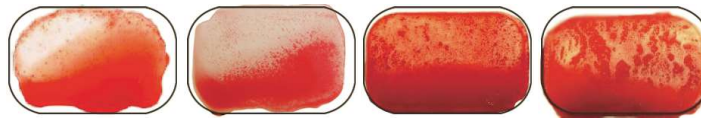

Example 1

Example 2

Example 3

Example 4

If the assay was run correctly, visible agglutination (see various positive examples above) should have occurred in the well marked Positive Control.

If the patient sample shows visible agglutination (see various positive examples above) in the well marked "Patient Test," the patient is DEA 1 positive. The "Auto-Agglutination Pre-Screen" well should not show agglutination. (Auto-agglutination is a rare occurrence.)

Negative for Agglutination



If no agglutination is visible in the well marked "Patient Test," (see negative example above), the patient is DEA 1 negative.

dmslaboratories, inc.

2 Darts Mill Road, Flemington, NJ 08822

Tel.: (908) 782-3353 or (800) 567-4367 in U.S. and Canada; Fax: (908) 782-0832
RapidVet is a registered trademark of **dms**laboratories, inc.